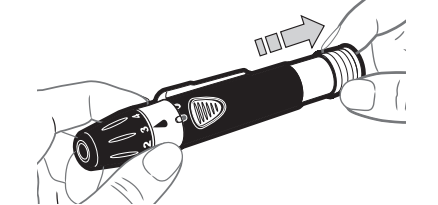
Dováža a distribuuje:
Bio G, spol. s r.o.
Elektrárenská 12092
831 04 Bratislava
Slovenská republika
Tel.: +421 2 4552 6381
E-Mail: bio@biog.sk
www.biog.sk
www.diabetes.ascensia.com

Ascensia Diabetes Care
Imported and distributed by:
Pharmaco Distribution (Pty) Ltd.
91 Protea Road
Chislehurst
Sandton, 2196
Johannesburg, South Africa
+27(11) 784 0077
www.diabetes.ascensia.com

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EN

INSTRUCTIONAL INSERT

- Lancet ejector for safe lancet disposal.
- Black/white adjustable endcap for fingertip testing only.
- Clear AST endcap for alternative site testing only.

Use with MICROLET® lancets for fingertip and palm testing.

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INTENDED USE: Intended for collecting capillary blood, for use together with disposable lancets. **The MICROLET NEXT lancing device** is used to draw capillary blood from the fingertip or palm for blood glucose testing with compatible lancets. The MICROLET NEXT lancing device is for single-patient use. The lancing device is not intended for neonates. **The MICROLET lancet** is a single-use sterile device intended for drawing a capillary blood sample from an adult or child for glucose monitoring. The lancet is not intended for neonatal use. There are no known contraindications associated with these devices.

WARNING

- Ask your health care professional if Alternative Site Testing (AST) is right for you.
- Do not calibrate a continuous glucose monitoring device from an AST result.
- Do not calculate an insulin dose based on an AST result.

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Do not use AST under the following conditions:

- If you think your blood glucose is low (hypoglycaemia).
- When blood glucose is changing rapidly (after a meal, insulin dose, or exercise).
- If you are unable to feel symptoms of low blood glucose (hypoglycaemic unawareness).
- If you get alternative site blood glucose results that do not agree with how you feel.
- During illness or times of stress.
- If you will be driving a car or operating machinery.

WARNING: Potential Biohazard

- The MICROLET NEXT lancing device is intended for self-testing by a single patient. It must not be used on more than one person due to the risk of infection.
- Do not reuse lancets. Used lancets are not sterile. Use a new lancet each time you test.

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- Always dispose of used test strips and lancets as medical waste or as advised by your health care professional.
- Always wash your hands with soap and water and dry them well before and after testing or handling the meter, lancing device, or test strips.

CAUTION:

- Contact Customer Service for assistance if your device malfunctions for any reason or if you notice any difficulties using the device.
- Choose alternate puncture sites to prevent excess scarring, which could lead to loss of sensitivity.
- Test your blood glucose level as frequently as recommended by your healthcare provider.*

*Healthcare agencies recommend that adult patients test their BGL three to four times per day (2018 U Ojha et al., "Disruption in the diabetic device care market") and that children and young persons test at least five times per day (2016 NICE [NG18], "Diabetes (type 1 and 2) in children and young people: diagnosis and management").

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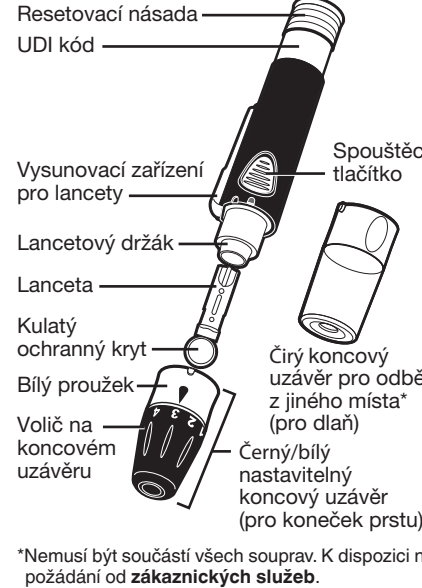
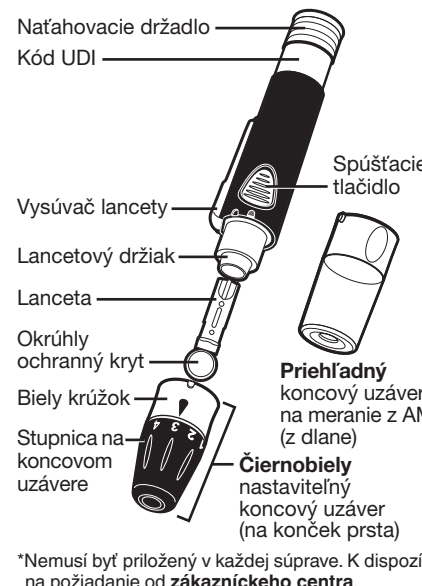
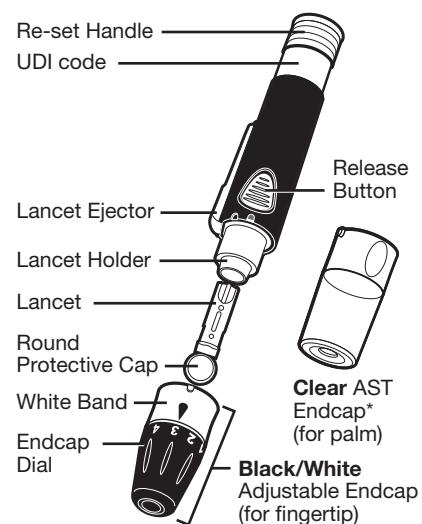
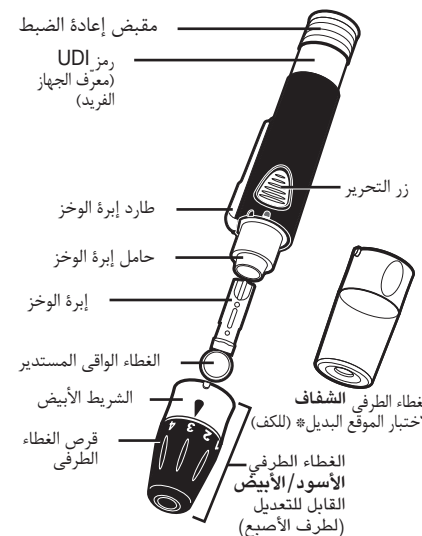
*May not be included in all kits. Available on request from **Customer Service**.

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Prepare the Lancing Device

WARNING

- Keep out of reach of children. This kit contains small parts that could cause suffocation if accidentally swallowed.
- Lancet remains sterile until the protective cap is removed. Do not use if the lancet is damaged or opened unintentionally. Dispose as medical waste to prevent any possible injury.

- Wash hands and the puncture site with soap and warm water. Rinse and dry well.
- Unlock the endcap. Twist the white band so the white raised mark and black indented mark line up as shown.

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3. Pull off the endcap.

4. Loosen the round protective cap on the lancet by rotating the cap ¼ turn, but do not remove the cap.

5. Insert the lancet firmly into the lancing device. Push until the lancet comes to a complete stop.

6. Carefully twist off the round protective cap. Save the cap for disposing of the used lancet.

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FINGERTIP TESTING

Use the **black/white** adjustable endcap. Do not use the clear AST endcap.

- Attach the **black/white** adjustable endcap.
- Look the endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

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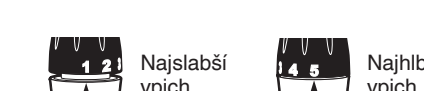
Ak nedôjde k vpichu, zopakujte 4. krok a zariadenie prestavte.

Pokračovanie na druhej strane

NOTE: The Capillary Sampling guidelines from the World Health Organization (WHO) recommend a depth for pediatric finger-prick sampling using a lancing device / lancet as follows:

- For a child over 6 months to 8 years: MICROLET NEXT Setting 1 or 2
- For a child over 8 years: MICROLET NEXT Setting 4

Settings should be adjusted based on comfort. Pressure applied can affect depth. (*WHO Guidelines on Drawing Blood: Best Practices in Phlebotomy*. Geneva: World Health Organization; 2010.)

Shallowest Puncture Deepest Puncture

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Ak nedôjde k vpichu, zopakujte 4. krok a zariadenie prestavte.

Pokračovanie na druhej strane

4. Pull back the re-set handle to prepare the lancing device. The release button is raised when the lancing device is primed.

5. Press the lancing device firmly against the side of the finger and press the release button until the device clicks.

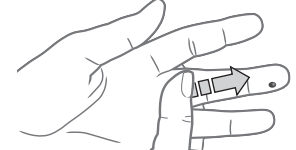
NOTE: Ensure you hear or feel the “click.”

If the device does not puncture, re-set by repeating Step 4.


Continued on reverse side

Imported and Distributed by:
Warba Medical Supplies Co.
Al Kharafti Tower
(Building 6), 1st Floor
Hamad Al Saqr street,
Qibla block 14
Kuwait
Telephone +965 22323856 / 57
Fax +965 22323864
Internet : www.warbamed.com
www.diabetes.ascensia.com

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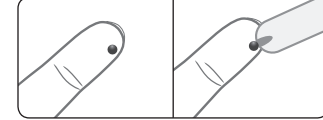
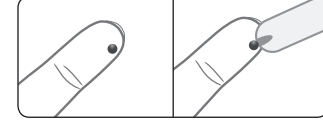
6. Stroke your hand and finger toward the puncture site to form a drop of blood. Do not squeeze around the puncture site.

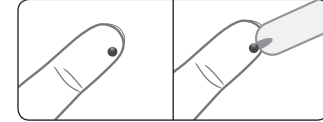7. Test immediately after you have formed a small, round blood drop, as shown.

8. Wipe the puncture site with a clean, dry tissue and **maintain pressure** on the site until the bleeding stops.

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Eject / Dispose of the Lancet

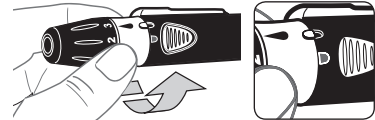
WARNING

Always dispose of the used lancet as medical waste to prevent injury or contamination to others. Use a new lancet each time you test. **Do not reuse lancets.**

Your MICROLET NEXT lancng device has a lancet ejector. Do not use your fingers to remove the lancet from the lancng device.

1. Carefully remove the endcap from the lancng device. Twist the white band so the white raised mark and the black indented mark line up. Pull off the endcap.

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“Click” to unlock

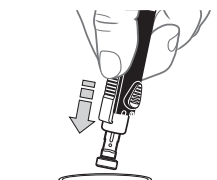
2. Place the round protective cap that you saved onto a flat surface. With the used lancet still in the lancng device, push the lancet into the center of the cap.

3. Point the device toward a container for sharp objects (an empty plastic bottle with a lid, for example). Label the container properly and keep the container out of the reach of children.

1. Carefully remove the endcap from the lancng device. Twist the white band so the white raised mark and the black indented mark line up. Pull off the endcap.

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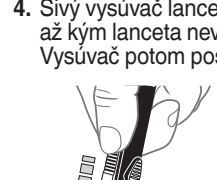
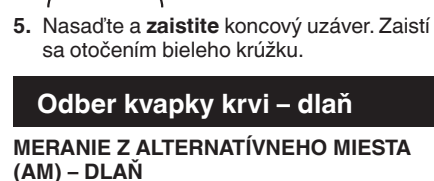
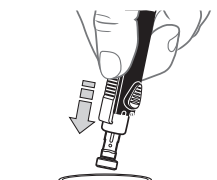
4. Push the gray lancet ejector forward until the lancet falls into the container, then slide ejector back.

5. Attach and **lock** the endcap. Twist the white band to lock.

Get the Blood Drop – Palm

ALTERNATIVE SITE TESTING (AST) PALM

NOTE: Use the **clear** AST endcap.

1. Insert the lancet as described in *Prepare the Lancng Device*.

2. Attach and lock the **clear** AST endcap. Twist the white band until you hear or feel a click and the white and black raised marks line up as shown.

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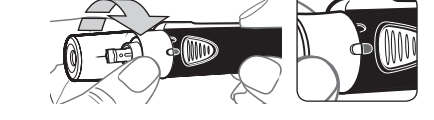
CAUTION: Do not use the **clear** AST endcap on your fingertips.

3. Select a puncture site from a fleshy area on the palm. Avoid veins, moles, bones, and tendons.

4. Pull back the re-set handle to prepare the lancng device. The release button is raised when the lancng device is ready.

5. Press the lancng device firmly against the puncture site, then press the release button until the device clicks.

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
If the device does not puncture, reset by repeating Step 4.

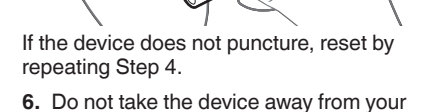6. Do not take the device away from your palm. Maintain steady pressure until a small, round blood drop forms under the clear endcap.

7. Lift the lancng device straight up and away from the skin without smearing the blood drop.

CAUTION: Do not test the blood sample from palm if you get:

• Smeared blood

• Runny blood

• Clotted blood

• Clear fluid mixed with blood

9. Wipe the puncture site with a clean, dry tissue and maintain pressure on the site until the bleeding stops.

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Signs of Deterioration: Your lancng device has been tested for 260 cycles of the cleaning and disinfection procedure (one cycle per week for 5 years). The lancng device has been demonstrated to withstand 5 years of cleaning and disinfection without damage. However, if your device malfunctions for any reason, contact Customer Service. See *Customer Service*.

CAUTION: Do not use bleach (sodium hypochlorite).

1. Clean the lancng device with 70% isopropyl alcohol until all soil is removed.

2. Allow to air dry.

3. Remove the endcap and eject the lancet, if present.

4. Get a new cloth dampened with 70% isopropyl alcohol and disinfect the lancng device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.

5. Allow the lancng device body and endcap to air dry.

WARNING
If your lancng device is being operated by a second person who is providing testing assistance to you, the second person should wear gloves and the lancng device should be disinfected prior to use to reduce the risk of transmitting blood-borne pathogens.

Signs of Deterioration: Your lancng device has been tested for 260 cycles of the cleaning and disinfection procedure (one cycle per week for 5 years). The lancng device has been demonstrated to withstand 5 years of cleaning and disinfection without damage. However, if your device malfunctions for any reason, contact Customer Service. See *Customer Service*.

CAUTION: Do not use bleach (sodium hypochlorite).

1. Clean the lancng device with 70% isopropyl alcohol until all soil is removed.

2. Allow to air dry.

3. Remove the endcap and eject the lancet, if present.

4. Get a new cloth dampened with 70% isopropyl alcohol and disinfect the lancng device body and endcap (inside and outside). For proper disinfection, you must wipe all surfaces and keep them wet for 2 minutes.

5. Allow the lancng device body and endcap to air dry.

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